


CERAMO® CONCEPT APART PUNCHES

1 Disassembly

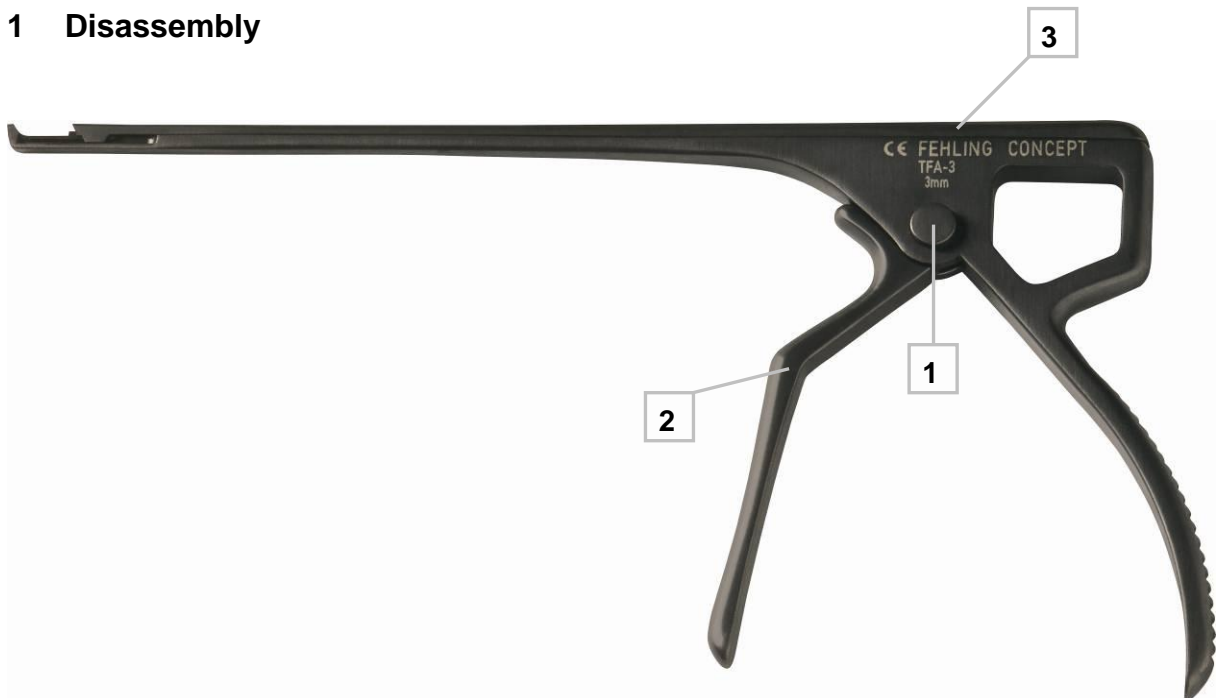

- Take the punch in both hands with the label showing upward.
- Press the push-button (no. 1) with the thumb of the right hand and place the index finger of the right hand on the slider (no. 3) at the same time (so that the slider cannot drop out uncontrollably).
- If the release button cannot be pressed further down, hold this position and at the same time pull the left handle part (no. 2) diagonally downward with the left hand.
- The slider (no. 3) comes off its holder and can be removed easily.
- Store and process both component parts in the same receptacle.



2 Assembly



- Place the slider with the spring (no. 4) on the milled groove (no. 5) in the shaft and at the same time place the front guide of the slider (no. 6) on the guide rail (no. 7) of the shaft.
- Slowly push the slider forward until the rear end of the slider projects app. 5 mm over the shaft end pointing towards the tip.
- Slide the left handle part diagonally upward. The handle locks in place with a click, and the push-button (no. 1) pops out in its working position.