


FEHLING BAR RETRACTOR WITH PINION

1) Components / Explanations


Example: MBT-4

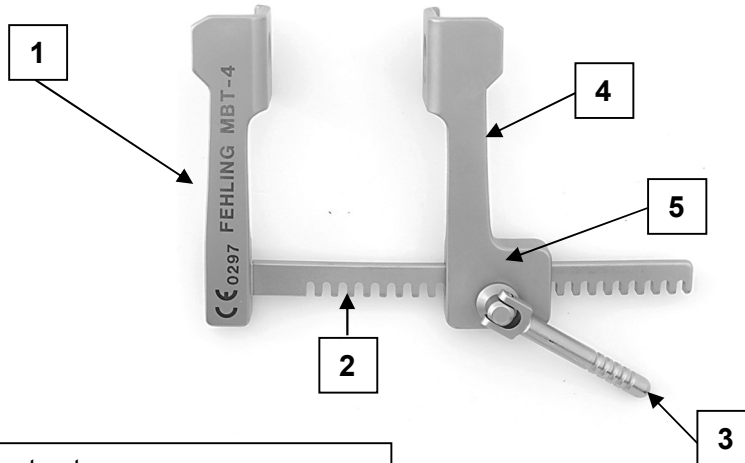

- 1. Fixed retractor arm
- 2. Toothed rack
- 3. Crank handle
- 4. Flexible retractor arm
- 5. Cage


2) Disassembly

1. Turn the crank handle (3) counterclockwise to advance the flexible retractor arm (4) outwards along the toothed rack (2) until it can be removed.
2. Remove the crank handle (3) in the second step.
3. The instrument is now disassembled into its three separate parts and can be re-processed.

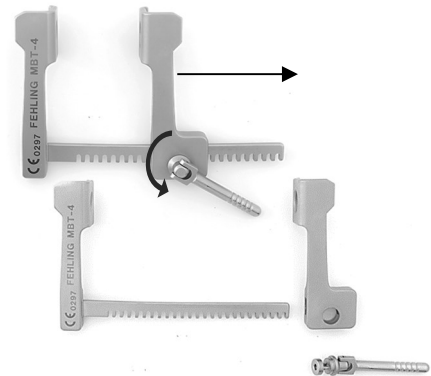


3) Assembly

1. First insert the crank handle (3) into the recess provided in the cage (5).

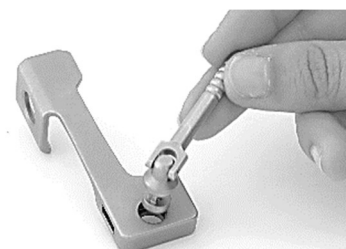


2. Insert the toothed rack (2) into the recess of the cage (5) until the pinion of the crank handle (3) engages the toothed rack (2).

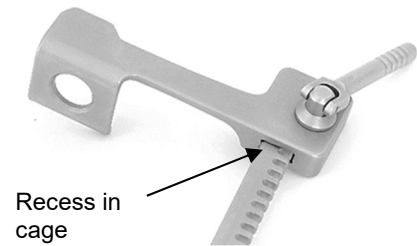





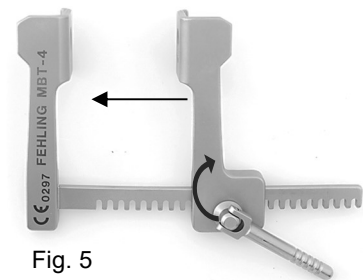




Ensure that both retractor arms point in the same direction as shown in Figure 5.

3. Turn the crank handle (3) clockwise to advance the flexible retractor arm (4) along the toothed rack (2) inwards in the direction of the fixed retractor arm (1).
4. Following a functional test, the assembled instrument is now ready for use again.



Symbols

In as far as depicted on the medical device or medical device label or instructions for use or instructions for assembly, the symbols represent the following meaning:

 Manufacturer	 Instructions for Use are to be observed	 Warning
 Article number	 Batch code	 Serial number
 CE labeling	 CE labeling	 Oil can for points to be lubricated

To contact the manufacturer:

	<p>FEHLING INSTRUMENTS GmbH & Co. KG Hanauer Landstr. 7A 63791 Karlstein, Germany Tel.: +49 (0) 6188-957440 Fax: +49 (0) 6188-957445 E-mail: info@fehling-instruments.de www.fehling-instruments.de</p>	
---	--	---