


FEHLING BLADE GUIDES (FOR BALL ADAPTERS) AND GUIDE FORCEPS

1a) Components / Explanations for two-part instruments


Example: MRN-3A

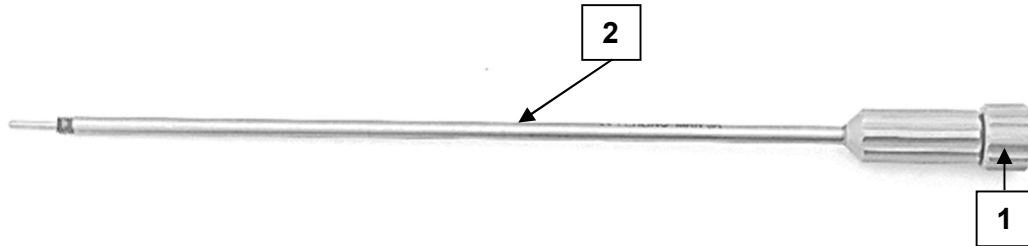

- 1. Inner rod with fixing nut
- 2. outer sleeve

1b) Components / Explanations for three-part instruments


Example: MRI-0

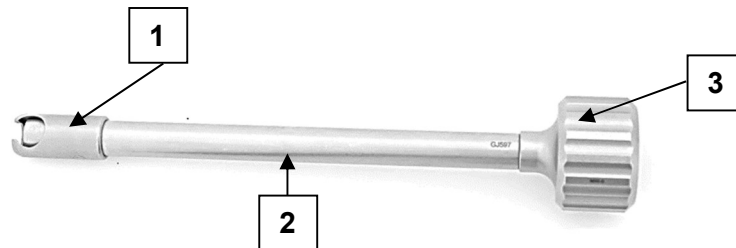


- 1. Inner rod
- 2. outer sleeve
- 3. Fixing nut

2a) Disassembly of two-part instruments

1. Rotate the fixing nut on the inner rod (1) counterclockwise until the thread is completely loosened and the inner rod can be pulled from the outer sleeve (2) in direction of the fixing nut.

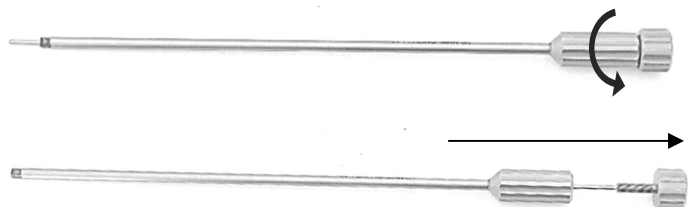

- The instrument is now disassembled into its separate parts and can be reprocessed.

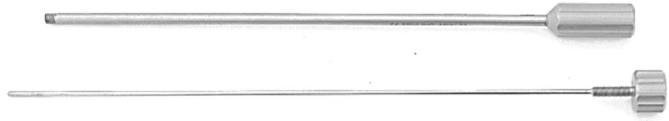


2b) Disassembly of three-part instruments


- Rotate the fixing nut (3) counterclockwise until it can be removed from the thread of the inner rod (1).



- Pull the outer sleeve (2) off the inner rod (1) in direction of the thread.

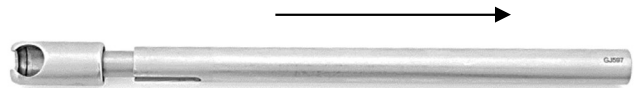

- The instrument is now disassembled into its separate parts and can be reprocessed.

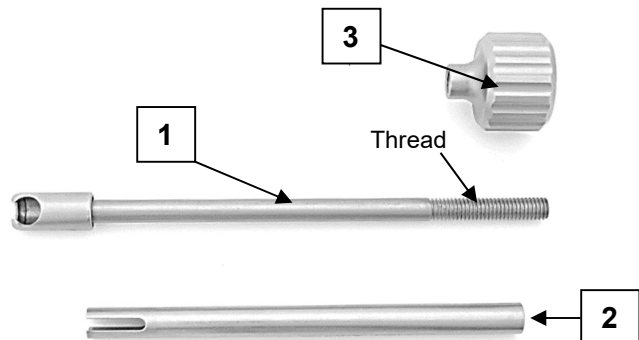

3a) Assembly two-part instrument

- Slide the outer sleeve (2) over the inner rod (1).




- Place the fixing nut (3) on the thread of the inner rod (1) and tighten clockwise.



- Following a functional test, the assembled instrument is now ready for use again.

3b) Assembly three-part instrument

- Slide the outer sleeve (2) over the thread onto the inner rod (1).

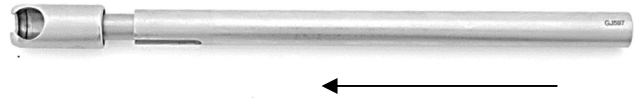

Make sure that the crossbar, which is located in the mount for the ball adapters of the inner rod (1), slides into the two slots of the outer sleeve (2).

- Place the fixing nut (3) on the thread of the inner rod (1) and tighten clockwise.



- Following a functional test, the assembled instrument is now ready for use again.

Symbols

In as far as depicted on the medical device or medical device label or instructions for use or instructions for assembly the symbols represent the following meaning:

 Manufacturer	 Instructions for Use are to be observed	 Warning
 Article number	 Batch code	 Serial number
 CE labeling	 CE labeling	 Oil can for points to be lubricated

To contact the manufacturer:

	<p>FEHLING INSTRUMENTS GmbH & Co. KG Hanauer Landstr. 7A 63791 Karlstein, Germany Tel.: +49 (0) 6188-9574-40 Fax: +49 (0) 6188-9574-45 E-mail: info@fehling-instruments.de www.fehling-instruments.de</p>	
--	--	--