


FEHLING VASCULAR RETRACTOR

1) Components / Explanations


EEV-1

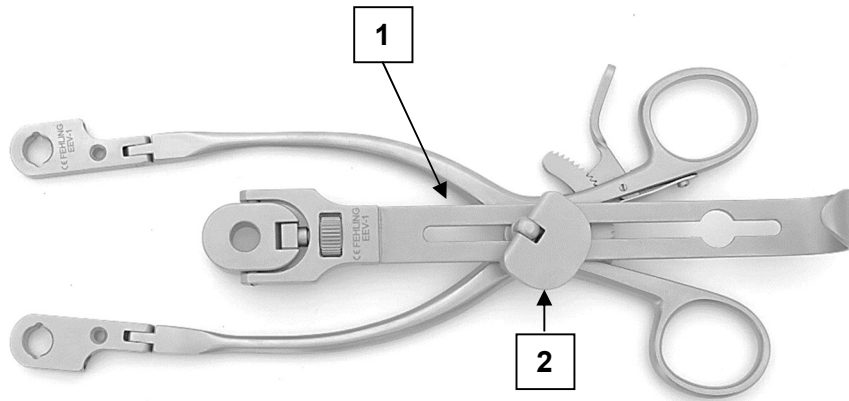

- 1. Blade guide
- 2. Wing nut


2) Disassembly

1. Turn the wing nut (2) counterclockwise until it can be removed. Then remove the blade guide (1).
2. The instrument is now disassembled into its separate parts and can be reprocessed.

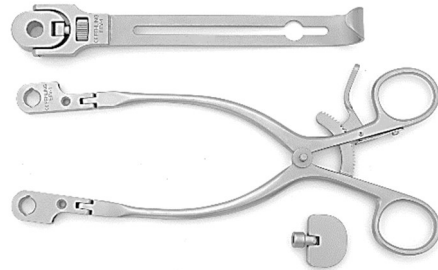


3) Assembly

1. Place the rail of the blade guide (1) on the mount.

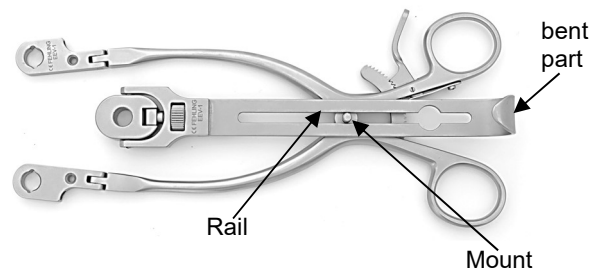

Ensure that the bent part of the blade guide points away from the mounting surface.

2. In the next step, place the wing nut (2) on the thread of the mount and tighten clockwise
3. Following a functional test, the assembled instrument is now ready for use again.

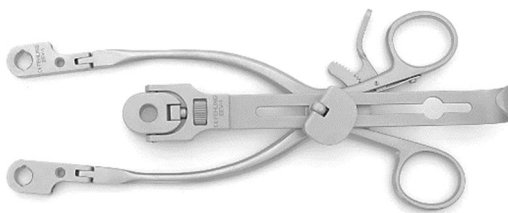

Symbols

In as far as depicted on the medical device or medical device label or instructions for use or instructions for assembly, the symbols represent the following meaning:

 Manufacturer	 Instructions for Use are to be observed	 Warning
 Article number	 Batch code	 Serial number
 CE labeling	 CE labeling	 Oil can for points to be lubricated

To contact the manufacturer

	FEHLING INSTRUMENTS GmbH & Co. KG Hanauer Landstr. 7A 63791 Karlstein, Germany Tel.: +49 (0) 6188-9574-40 Fax: +49 (0) 6188-9574-45 E-mail: info@fehling-instruments.de www.fehling-instruments.de	
--	---	--