


### FEHLING CANNULAS AND SUCTION DEVICES WITH REMOVABLE BASKET

Applies to all cannulas and suction devices with removable basket

#### 1) Components / Explanations

Example: MNM-3

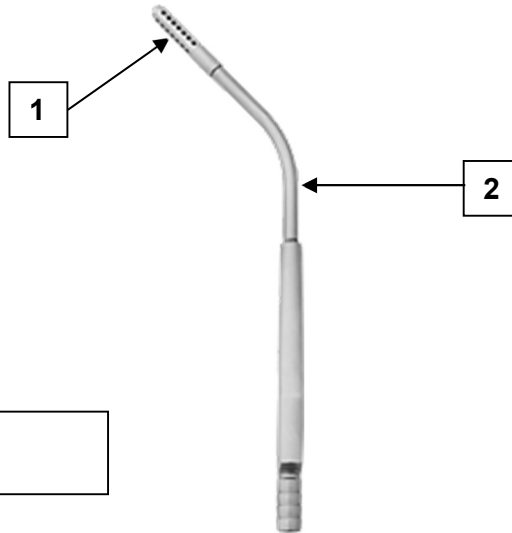


- 1. Basket
- 2. Suction device/cannula

#### 2) Disassembly

1. Rotate the basket (1) on the front part of the suction device or cannula (2) counterclockwise until it can be removed from the thread.


2. The instrument is now disassembled into its separate parts and can be reprocessed.

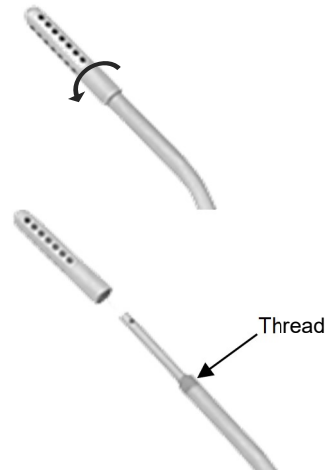







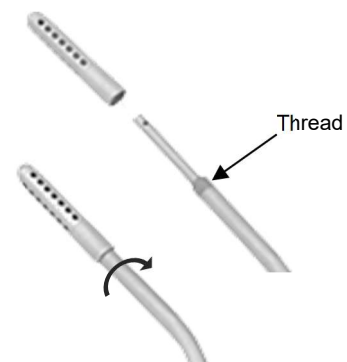




#### 3) Assembly

1. Place the basket (1) on the thread on the front part of the suction device or cannula (2) and tighten clockwise.

2. Following a functional test, the assembled instrument is now ready for use again.



Symbols		
In as far as depicted on the medical device or medical device label or instructions for use or instructions for assembly, the symbols represent the following meaning:		
 Manufacturer	 Instructions for Use are to be observed	 Warning
 Article number	 Batch code	 Serial number
 CE labeling	 CE labeling	 Oil can for points to be lubricated
To contact the manufacturer		
	FEHLING INSTRUMENTS GmbH & Co. KG Hanauer Landstr. 7A 63791 Karlstein, Germany Tel.: +49 (0) 6188-9574-40 Fax: +49 (0) 6188-9574-45 E-mail: <a href="mailto:info@fehling-instruments.de">info@fehling-instruments.de</a> <a href="http://www.fehling-instruments.de">www.fehling-instruments.de</a>	