

### SOMOHA RETRACTOR

#### 1) Components / Explanations

Exemplary: MTJ-3

Assembled condition



Disassembled condition




- 1. Guide tabs
- 2. Insertion opening
- 3. Guide bar

#### 2) Disassembly




Disassembling the retractor requires a certain amount of force due to the nature of the material.

1. To disassemble the retractor, take it in both hands and push the guide tabs (1) towards the insertion opening (2).

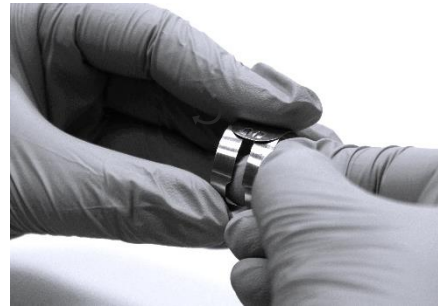

Ensure that pressure is applied evenly from both sides so that the guide tabs (1) do not tilt.

2. Release the guide tabs (1) from the guide bar (3) via the insertion opening (2). The retractor can then be rolled out.


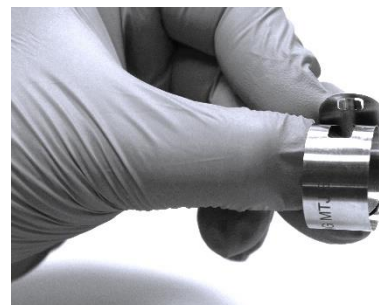


### 3) Assembly




The assembly of the retractor requires a certain amount of force due to the nature of the retractor.

1. For assembly, bring both ends of the retractor on top of each other.  
The end with the guide tabs (1) must be pushed over the end with the guide bar (3).

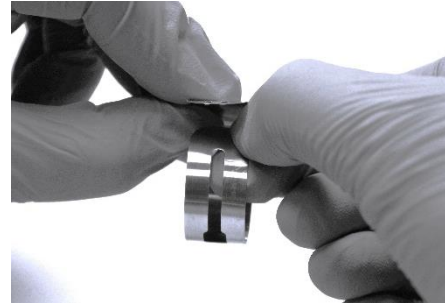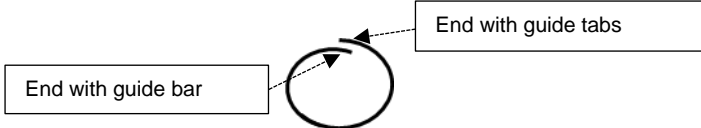

2. Place the guide tabs (1) over the insertion opening (2) and insert the guide tabs (1).  
Then release the pressure slightly and allow the guide tabs (1) to slide along the guide bar (3).

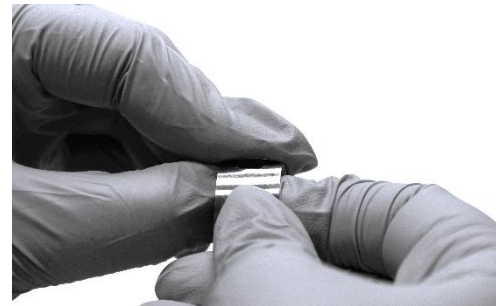














If not enough pressure is applied, the retractor may "jump" back into the disassembled form.

3. The assembled instrument is now ready for use again after a functional test.



After assembly, check that both guide tabs (1) are engaged in the insertion opening (2).

Symbols		
Where shown on the medical device or medical device label or instructions for use or assembly instructions, the symbols have the following meaning in accordance with DIN EN ISO 15223-1:		
 Manufacturer	 Unique Device identifier	 Caution
 Catalog number	 Batch code	 Serial number
 Medical device	 CE marking	 CE marking
 Oil can for points to be lubricated	 Consult instructions for use or consult electronic instructions for use	
Contact the manufacturer		 CE marking
	FEHLING INSTRUMENTS GmbH & Co KG Hanauer Landstr. 7A 63791 Karlstein/Germany Phone: +49 (0) 6188-9574-40 Fax: +49 (0) 6188-9574-45 E-mail: info@fehling-instruments.de www.fehling-instruments.de	