


### FEHLING HANDLE WITH SCREW SLEEVE

#### Article numbers

ABB-1, ABB-1F, ABB-2, ABB-3, ABB-5, ABB-6

#### 1) Components / Explanations

- 1. Screw sleeve, knurled
- 2. Handle, knurled

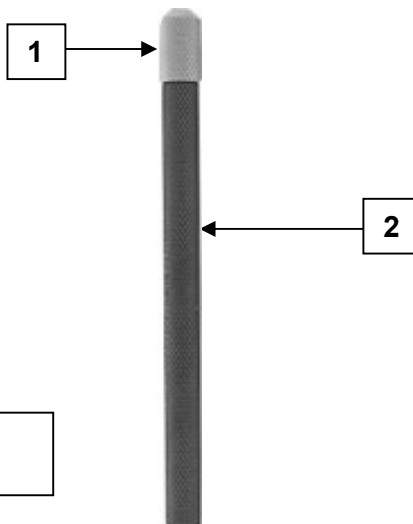

#### 2) Disassembly

1. Rotate the screw sleeve (1) on the front part of the handle (2) counterclockwise until it can be removed from the thread.




2. The instrument is now disassembled into its separate parts and can be reprocessed.



To change or remove the shaft from the handle (2), the screw sleeve (1) only needs to be loosened.

### 3) Assembly

1. Place the screw sleeve (1) on the thread on the front part of the handle (2) and tighten clockwise.

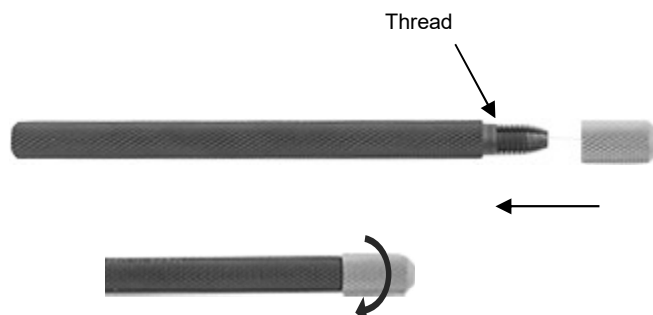










2. Following a functional test, the assembled instrument is now ready for use again.



To change or insert the shaft into the handle (2), the screw sleeve (1) only needs to be loosened.

### Symbols

In as far as depicted on the medical device or medical device label or instructions for use or instructions for assembly, the symbols represent the following meaning:

 Manufacturer	 Instructions for Use are to be observed	 Warning
 Article number	 Batch code	 Serial number
 CE labeling	 CE labeling	 Oil can for points to be lubricated

### To contact the manufacturer

	FEHLING INSTRUMENTS GmbH & Co. KG Hanauer Landstr. 7A 63791 Karlstein, Germany Tel.: +49 (0) 6188-9574-40 Fax: +49 (0) 6188-9574-45 E-mail: <a href="mailto:info@fehling-instruments.de">info@fehling-instruments.de</a> <a href="http://www.fehling-instruments.de">www.fehling-instruments.de</a>	
---	---	---