


FEHLING ASPIRATOR WITH REMOVABLE LUER-LOCK CONNECTION (LL CONNECTION)

Valid for all aspirators with detachable LL connection

1) Components / basis for explanation



Example: MRK-5

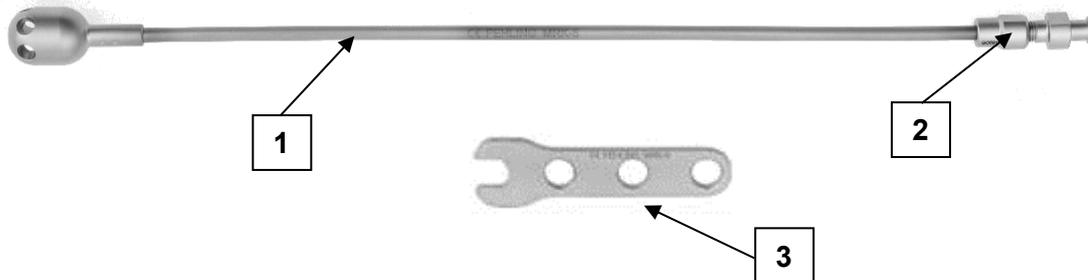

- 1 Aspirator
- 2 Removable LL connection
- 3 Open-end wrench (e.g. MRK-6 counter wrench)


2) Disassembly

1. To remove, hold the LL connection (2) using two open-ended wrenches (3) and loosen the union nut on the inside by turning it anti-clockwise.
2. Remove the LL connection (2) from the sleeve of the aspirator (1).
3. The LL connection (2) consists of two parts, a slotted mounting thread and a union nut. To separate the two parts, turn the union nut anticlockwise until it can be removed.
4. The LL connection, which has been disassembled into its individual parts, can now be reprocessed.

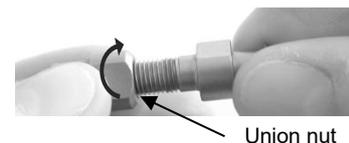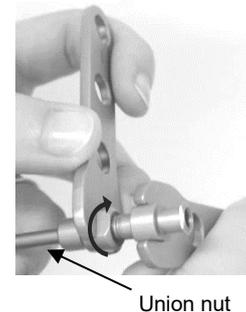

3) Assembly

1. To install, slide the union nut of the LL connection (2) over the sleeve of the aspirator (1).
2. Also place the slotted mounting thread of the LL connection (2) on the sleeve and push it as far as it will go.
3. Slide the union nut onto the slotted mounting thread and turn clockwise as far as it will go using two open-end wrenches (3).
4. The LL connection (2) is now firmly reconnected to the aspirator and ready for use again after a function test.

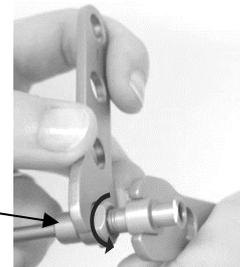

Union nut



Symbols		
If displayed on the medical device, the label of the medical device or the instructions for use, the symbols according to DIN EN ISO 15223-1 have the following meaning:		
 Manufacturer	 Consult instructions for use or consult electronic instructions for use	 Caution
 Catalog number	 Batch code	 Serial number
 Medical device	 Unique Device Identifier	 CE marking
 Oil can for points to be lubricated	 CE marking	
Contact the manufacturer		
	FEHLING INSTRUMENTS GmbH Seligenstädter Str. 100 D-63791 Karlstein/Germany Tel.: +49 (6188) 9574-40 Fax: +49 (6188) 9574-45 E-Mail: info@fehling-instruments.de www.fehling-instruments.de	