


FEHLING CHORDAE GAUGE AND KNOT ASSISTANT

MNV-4, MNV-7, MNV-9



1) Components / basis for explanation

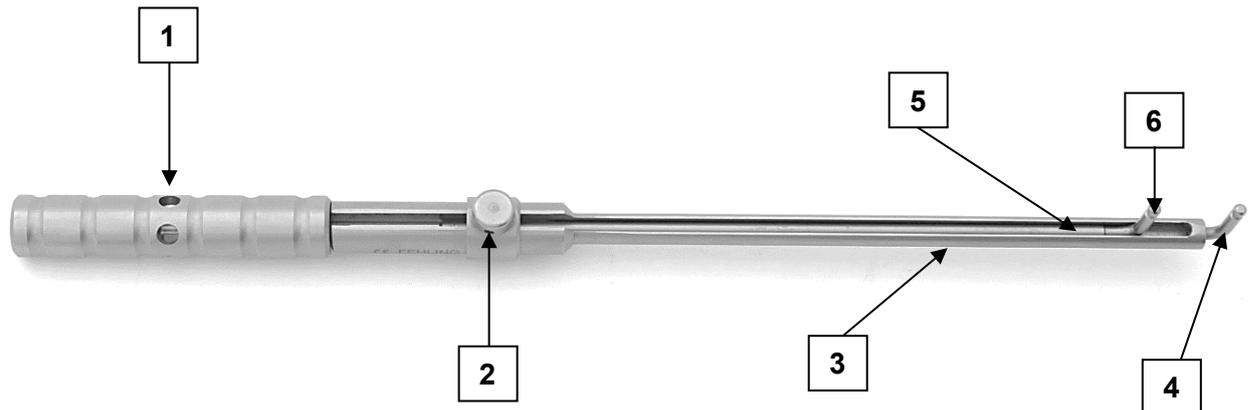

- 1 Instrument handle
- 2 Release button
- 3 Instrument shaft
- 4 Working end shaft
- 5 Instrument slider
- 6 Working end slider


2) Disassembly

1. Turn the instrument handle (1) counterclockwise until the handle (1) is completely released.
2. In the next step, remove the slider (5). To do this, press the release button (2) in direction of the shaft and at the same time withdraw the slider (5) fully in direction of the shaft end.
3. The instrument has now been disassembled into its three components and can be reprocessed.

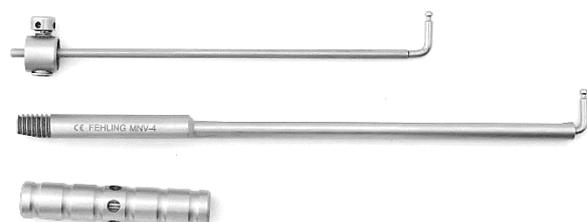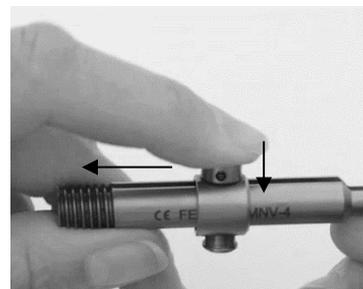

3) Assembly

1. Insert the slider (5) into the end of the shaft beyond the thread in direction of the working end (4). At the same time, press the release button (2) in direction of the shaft.

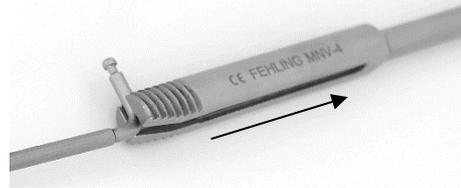

Ensure that the working ends of the instrument slider (5) and instrument shaft (3) point in the same direction.

2. Finally, turn the handle (1) clockwise onto the thread of the shaft (3).



3. Following a functional test, the assembled instrument is now ready for use again.



Symbols		
If displayed on the medical device, the label of the medical device or the instructions for use, the symbols according to DIN EN ISO 15223-1 have the following meaning:		
 Manufacturer	 Consult instructions for use or consult electronic instructions for use	 Caution
 Catalog number	 Batch code	 Serial number
 Medical device	 Unique Device Identifier	 CE marking
 Oil can for points to be lubricated	 CE marking	
Contact the manufacturer		
	FEHLING INSTRUMENTS GmbH Seligenstädter Str. 100 D-63791 Karlstein/Germany Tel.: +49 (6188) 9574-40 Fax: +49 (6188) 9574-45 E-Mail: info@fehling-instruments.de www.fehling-instruments.de	