


FEHLING BAR RETRACTOR WITH SPROCKET / LOCK

Valid for all bar retractors with sprocket (lock)

1) Components / Explanations



Example: MRP-1

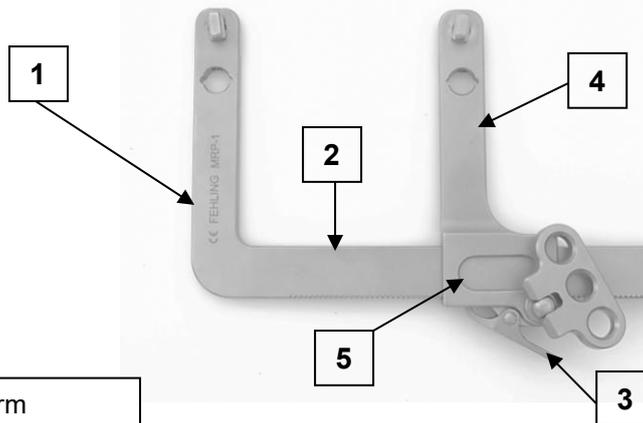

- 1 Fixed retractor arm
- 2 Toothed rack
- 3 Lock
- 4 Flexible retractor arm
- 5 Cage


2) Disassembly

1. Advance the flexible retractor arm (4) outwards along the toothed rack (2) until it can be removed. At the same time, release the lock (3) by pressing in the direction of the toothed rack (2).
2. The instrument is now disassembled into its separate parts and can be reprocessed.

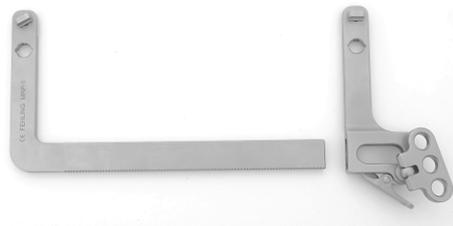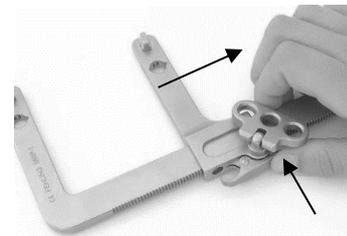


3) Assembly

1. Insert the toothed rack (2) into the recess of the cage (5). At the same time, release the lock (3) by pressing in the direction of the toothed rack (2).

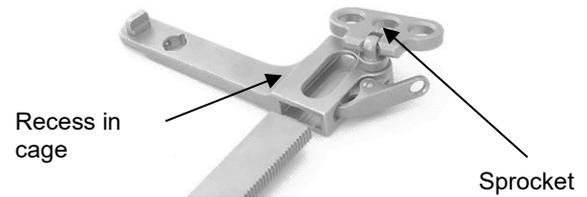


Ensure that both retractor arms point in the same direction and the sprocket of the flexible retractor arm (4) points outwards.

2. Advance the flexible retractor arm (4) along the toothed rack (2) inwards in the direction of the fixed retractor arm (1).

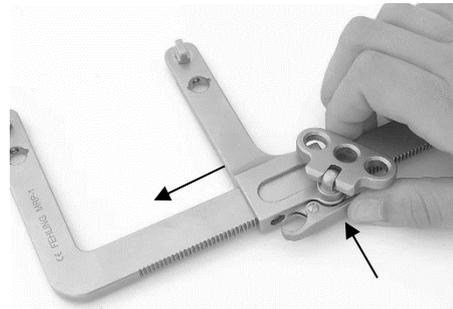

3. Following a functional test, the assembled instrument is now ready for use again.

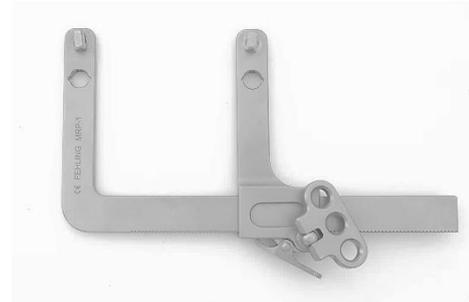

Symbols		
If displayed on the medical device, the label of the medical device or the instructions for use, the symbols according to DIN EN ISO 15223-1 have the following meaning:		
 Manufacturer	 Consult instructions for use or consult electronic instructions for use	 Caution
 Catalog number	 Batch code	 Serial number
 Medical device	 Unique Device Identifier	 CE marking
 Oil can for points to be lubricated	 CE marking	
Contact the manufacturer		
	FEHLING INSTRUMENTS GmbH Seligenstädter Str. 100 D-63791 Karlstein/Germany Tel.: +49 (6188) 9574-40 Fax: +49 (6188) 9574-45 E-Mail: info@fehling-instruments.de www.fehling-instruments.de	