


FEHLING OPERATING TABLE ADAPTING CLAMP

EEJ-1, EEK-1F, EEK-1S

1) Components / Explanations

Example: EEJ-1

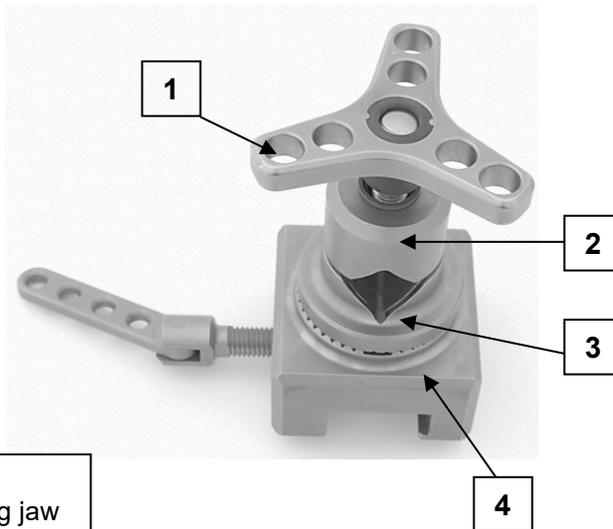


- 1 Star grip
- 2 Upper part of the clamping jaw
- 3 Lower part of the clamping jaw
- 4 Base

2) Disassembly

1. Turn the star grip (1) counterclockwise until it can be removed.




2. First pull the upper part of the clamping jaw (2) from the thread of the base (4), then remove the lower part of the clamping jaw (3).

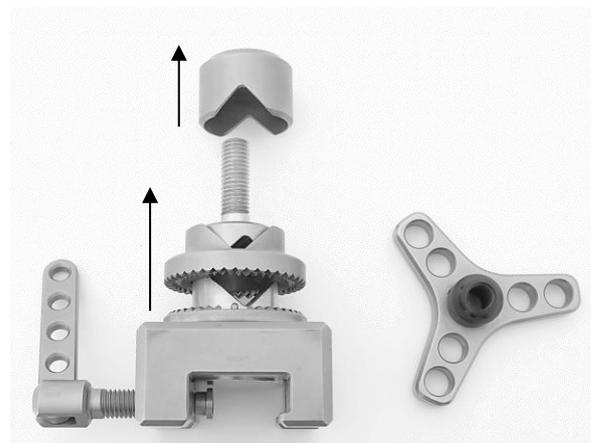

3. The instrument, disassembled into its individual parts, can now be reprocessed.

3) Assembly

1. Place the lower part of the clamping jaw (3) on the base (4), then insert the upper part of the clamping jaw (2) over the thread of the base (4).

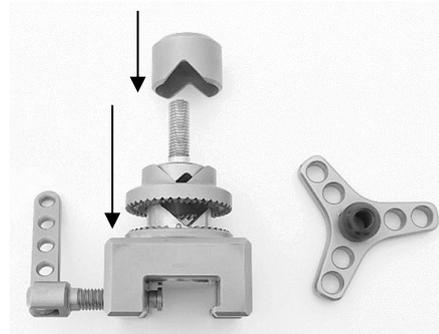

2. Place the star grip (1) on the thread of the base (4) and tighten clockwise.



3. Following a functional test, the assembled instrument is now ready for use again.

Symbols

If displayed on the medical device, the label of the medical device or the instructions for use, the symbols according to DIN EN ISO 15223-1 have the following meaning:

 Manufacturer	 Consult instructions for use or consult electronic instructions for use	 Caution
 Catalog number	 Batch code	 Serial number
 Medical device	 Unique Device Identifier	 CE marking
 Oil can for points to be lubricated	 CE marking	

Contact the manufacturer	
	<p>FEHLING INSTRUMENTS GmbH Seligenstädter Str. 100 D-63791 Karlstein/Germany Tel.: +49 (6188) 9574-40 Fax: +49 (6188) 9574-45 E-Mail: info@fehling-instruments.de www.fehling-instruments.de</p>
	