


FEHLING CANNULAS AND SUCTION DEVICES WITH REMOVABLE BASKET

Applicable to all cannulas and suction devices with removable basket

1) Components / Explanations

Example: MNM-3

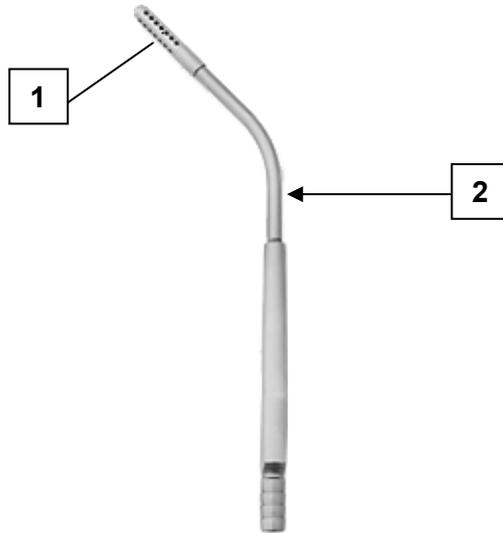

- 1 Basket
- 2 Suction device/cannula

2) Disassembly

1. Rotate the basket (1) on the front part of the suction device or cannula (2) counterclockwise until it can be removed from the thread.




2. The instrument is now disassembled into its separate parts and can be re-processed.



3) Assembly

1. Place the basket (1) on the thread on the front part of the suction device or cannula (2) and tighten clockwise.

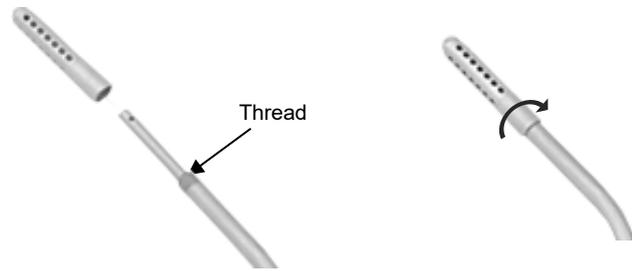

2. Following a functional test, the assembled instrument is now ready for use again.



Symbols

If displayed on the medical device, the label of the medical device or the instructions for use, the symbols according to DIN EN ISO 15223-1 have the following meaning:

 Manufacturer	 Consult instructions for use or consult electronic instructions for use	 Caution
 Catalog number	 Batch code	 Serial number
 Medical device	 Unique Device Identifier	 CE marking
 Oil can for points to be lubricated	 CE marking	
Contact the manufacturer		
	FEHLING INSTRUMENTS GmbH Seligenstädter Str. 100 D-63791 Karlstein/Germany Tel.: +49 (6188) 9574-40 Fax: +49 (6188) 9574-45 E-Mail: info@fehling-instruments.de www.fehling-instruments.de	