


FEHLING HANDLE WITH SCREW SLEEVE


ABB-1, ABB-1F, ABB-2, ABB-3, ABB-5, ABB-6

1) Components / Explanations

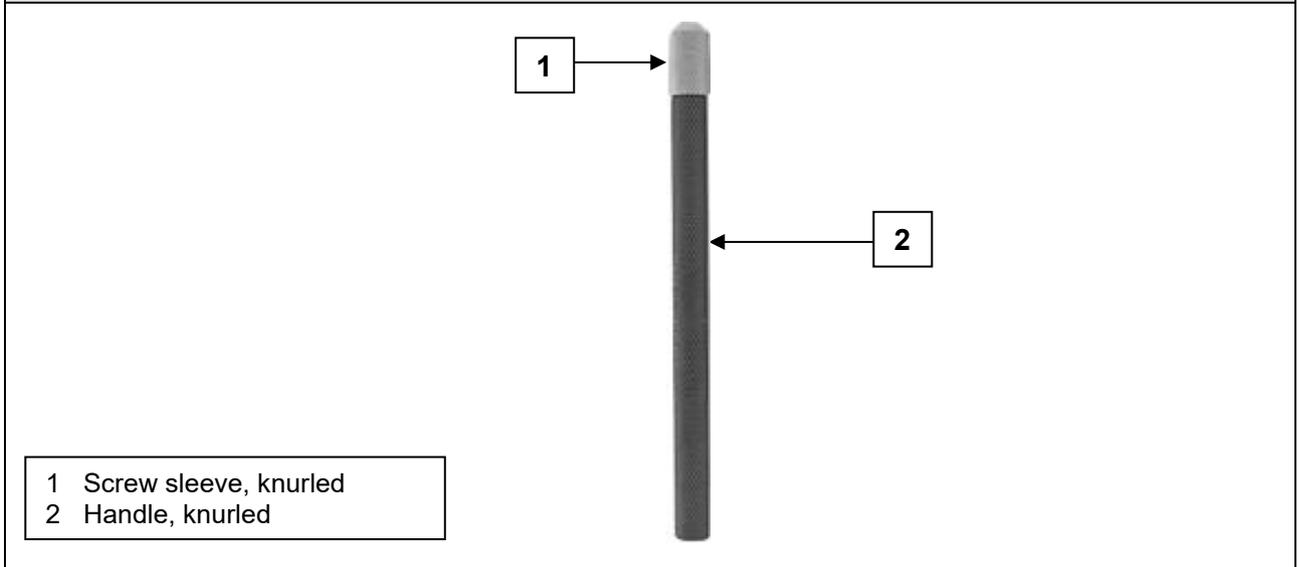

2) Disassembly

1. Rotate the screw sleeve (1) on the front part of the handle (2) counterclockwise until it can be removed from the thread.

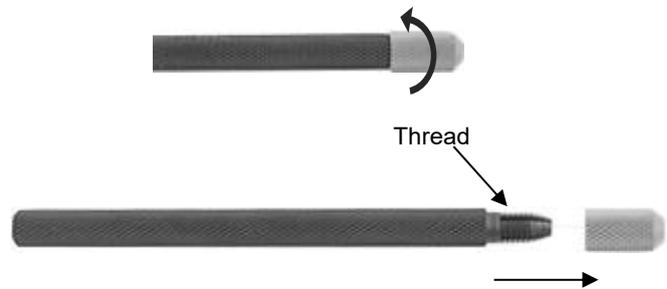


2. The instrument is now disassembled into its separate parts and can be reprocessed.



To change or remove the shaft from the handle (2), the screw sleeve (1) only needs to be loosened.

3) Assembly

1. Place the screw sleeve (1) on the thread on the front part of the handle (2) and tighten clockwise.

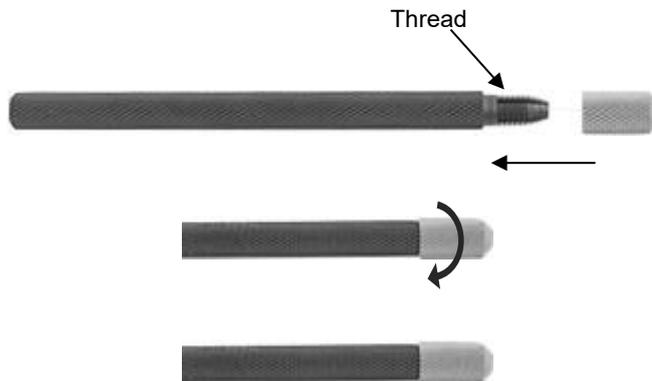

2. Following a functional test, the assembled instrument is now ready for use again.



To change or insert the shaft into the handle (2), the screw sleeve (1) only needs to be loosened.

Symbols

If displayed on the medical device, the label of the medical device or the instructions for use, the symbols according to DIN EN ISO 15223-1 have the following meaning:

 Manufacturer	 Consult instructions for use or consult electronic instructions for use	 Caution
 Catalog number	 Batch code	 Serial number
 Medical device	 Unique Device Identifier	 CE marking
 Oil can for points to be lubricated	 CE marking	
Contact the manufacturer		
	<p>FEHLING INSTRUMENTS GmbH Seligenstädter Str. 100 D-63791 Karlstein/Germany Tel.: +49 (6188) 9574-40 Fax: +49 (6188) 9574-45 E-Mail: info@fehling-instruments.de www.fehling-instruments.de</p>	
		