


FEHLING BLADE GUIDES (FOR BALL ADAPTERS) AND GUIDE CLAMPS

1a) Components / Explanations for two-part instruments


Example: MRN-3A

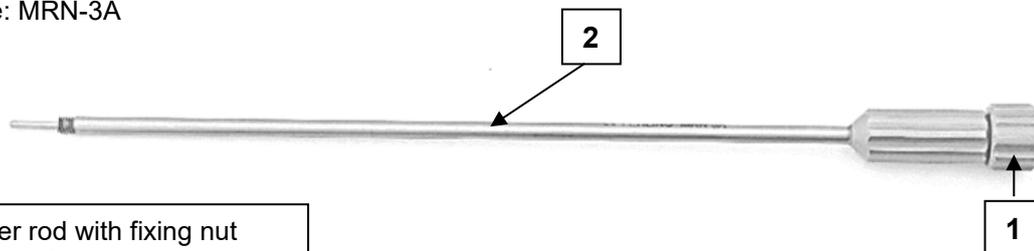

- 1 Inner rod with fixing nut
- 2 Outer sleeve

1b) Components / Explanations for three-part instruments


Example: MRI-0

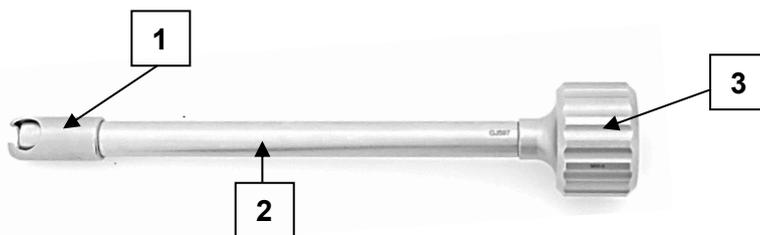


- 1 Inner rod
- 2 Outer sleeve
- 3 Fixing nut

2a) Disassembly of two-part instruments

1. Rotate the fixing nut (3) on the inner rod (1) counterclockwise until the thread is completely loosened and the inner rod (1) can be pulled from the outer sleeve (2) in direction of the fixing nut (3).

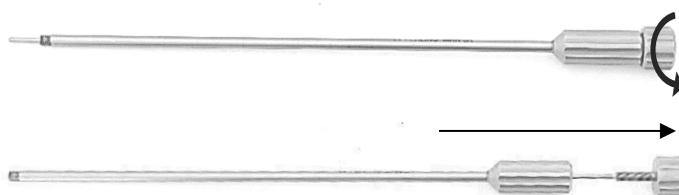

2. The instrument is now disassembled into its separate parts and can be re-processed.

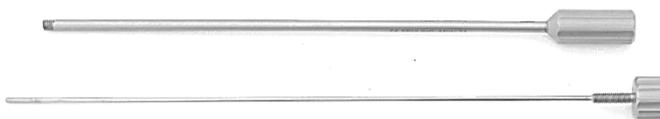


2b) Disassembly of three-part instruments


1. Rotate the fixing nut (3) counter-clockwise until it can be removed from the thread of the inner rod (1).



2. Pull the outer sleeve (2) off the inner rod (1) in the direction of the thread.

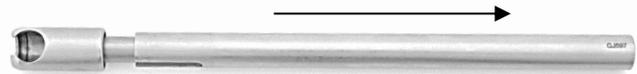


3. The instrument is now disassembled into its separate parts and can be re-processed.

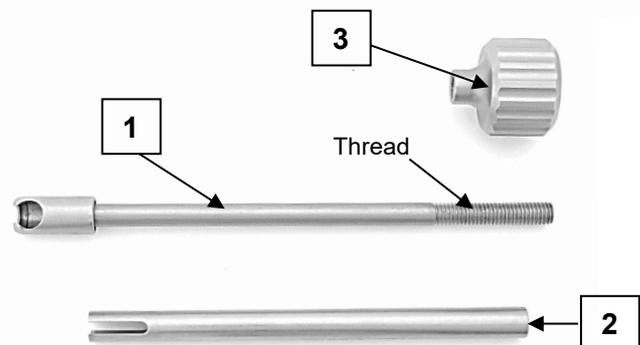

3a) Assembly of two-part instruments

1. Slide the outer sleeve (2) over the inner rod (1).

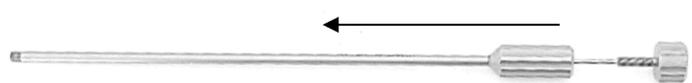

2. Place the fixing nut (3) on the thread of the inner rod (1) and tighten clockwise.



3. Following a functional test, the assembled instrument is now ready for use again.



3b) Assembly of three-part instruments

1. Slide the outer sleeve (2) over the thread onto the inner rod (1).



Make sure that the crossbar, which is located in the mount for the ball adapters of the inner rod (1), slides into the two slots of the outer sleeve (2).

2. Place the fixing nut (3) on the thread of the inner rod (1) and tighten clockwise.



3. Following a functional test, the assembled instrument is now ready for use again.



Symbols

If displayed on the medical device, the label of the medical device or the instructions for use, the symbols according to DIN EN ISO 15223-1 have the following meaning:

 Manufacturer	 Consult instructions for use or consult electronic instructions for use	 Caution
 Catalog number	 Batch code	 Serial number
 Medical device	 Unique Device Identifier	 CE marking
 Oil can for points to be lubricated	 CE marking	

Contact the manufacturer

	<p>FEHLING INSTRUMENTS GmbH Seligenstädter Str. 100 D-63791 Karlstein/Germany Tel.: +49 (6188) 9574-40 Fax: +49 (6188) 9574-45 E-Mail: info@fehling-instruments.de www.fehling-instruments.de</p>	 CE marking
---	--	---