



FEHLING Clamps for vessels, hollow organs and tissue



This instrument or medical device is non-sterile when delivered. It is to be reprocessed before use. The instrument must undergo risk assessment according to the RKI Guidelines (non-critical, semi-critical, critical A/B/C) before reprocessing.

The clamps may only be used, reprocessed and disposed of by qualified medical personnel.

The clamps are intended for re-use.

1) Intended purpose

Clamps are used for the temporary occlusion of vessels and hollow organs and for firmly grasping and holding tissue.

Additional information regarding the intended purpose

Duration of application: Clamps are intended for short-term use.

Field of application: Clamps are used in all patients in whom vessels and hollow organs must be temporarily occluded and tissue grasped and held.

User profile: Clamps may only be used by medically trained personnel (e.g. specialist physician).

Application environment: Clamps are only to be used in controlled environments (e.g. in the operating room).

Target patient population: No restrictions.

2) Indications

Any surgical procedure in which vessels and hollow organs must be temporarily occluded and in which tissue must be firmly grasped and held.

3) Contraindication

All applications that run counter to the physical and/or mechanical properties of the individual clamp model are contraindicated. There are no generally applicable contraindications for the use of clamps.

Nevertheless, increased risks must be taken into account which could result from the anatomical and physiological conditions as well as the clinical picture of the patient.



4) Possible side effects

The following side effects are described in the medical literature and may also occur during the intended use of clamps:

- Infections
- Lesions of surrounding structures (tissue, nerves, vessels)
- Risk of ischaemia from clamping structures for too long
- Necrosis
- Damage to the endothelium
- Perforation of tissue, vessels and cavities
- Detachment of plaques from the vessel intima/plaque rupture
- Secondary haemorrhage
- Blood clots (thrombosis or embolism)



Medical devices may, for example, contain chromium, nickel and/or titanium. The materials used are biocompatible, however they may cause allergic reactions or incompatibilities.



The vessels to be clamped should be palpated before using the vascular clamps to find a suitable location without calcified rings. This can minimise the risk of plaque rupture or vascular injury in most cases.

5) Before use

The clamps are non-sterile when delivered, so they must be cleaned and sterilised by the user before initial use and every time before they are used thereafter (see section 6) *Reprocessing*).



Perform a safety check before each use. Check for sharp edges, cracks, fractures or mechanical malfunctions and missing components (see section 6) *Reprocessing* under "*Maintenance, Inspection and Testing*").



The clamps must be handled with care during storage, transportation and cleaning. Avoid striking the clamps or applying pressure to their parts so as not to cause any consequential damage. Do not overstrain functional parts.



Always keep clamps with a locking mechanism in a released state. This counteracts premature fatigue of the spring tension.



Use only sterilised products of sound quality.




6) Reprocessing	
	The medical device is to be reprocessed before use. It must undergo risk assessment according to the RKI Guidelines (non-critical, semi-critical, critical A/B/C) before reprocessing.
	The national legal regulations, national and international standards and guidelines as well as the company's own hygiene regulations for reprocessing are to be complied with.
	The applicable national regulations must be followed for the reprocessing of instruments used in patients with Creutzfeldt-Jakob disease (CJD), suspected CJD or possible variants.
	The instruments may only be used, reprocessed and disposed of by qualified medical personnel.
	The instruments must be handled with care during storage, transportation and cleaning. Avoid striking the instruments or applying pressure to their parts so as not to cause any consequential damage. Do not overstrain functional parts.
	Always keep micro-instruments separate from the general instrument set and do not clean them together with other instruments in the washer-disinfector if possible. Protect micro-instruments from uncontrolled changes in position in the basket to avoid deformation/breakage. Risk of injury.
Limitations on reprocessing	Frequent reprocessing has little effect on the labelling of the instruments and does not impair their function. The end of product life is normally determined by wear and tear and damage occurring through use (e.g. damage, illegible marking, functional failure - also see " <i>Maintenance, Inspection and Testing</i> "). If used and reprocessed correctly, the instruments can demonstrably undergo at least 500 reprocessing cycles.
General information on reprocessing	Reprocessing is based on a validated procedure. All the cleaning steps mentioned (manual pre-cleaning, automated/manual cleaning, manual disinfection, and sterilisation) were validated with the respective parameters specified in each case and listed under "Validated procedure". The recommended reprocessing agents (detergent: Neodisher® MediClean forte (Dr. Weigert); disinfectant: Korsolex® med AF (Bode Chemie GmbH)) were used for validation. Both water of potable water quality and fully deionised water (deionised water, demineralised, microbiologically at least of potable water quality) are used for cleaning. Automated reprocessing is preferable to manual cleaning due to a better and safer cleaning result. There is also the option of cleaning our instruments with other tested and approved chemicals which have been recommended by the chemical manufacturer with regard to their material compatibility. Please always observe the manufacturer's information on concentration, exposure time, temperature and renewal of the detergents and disinfectants. All of the chemical manufacturer's instructions for use must be strictly observed. Otherwise, this can lead to visual material changes or material damage, such as, for example, corrosion, fractures or premature ageing.



<p>Pre-treatment at the place of use</p>	<p>Pre-cleaning: ensure that blood, tissue and drug residues are removed from the instruments with a disposable cloth/paper wipe immediately after completion of the procedure and that they undergo mechanical cleaning immediately. After completion of initial treatment of the instruments, visual inspections must be performed to ensure that the instruments are complete.</p> <p>The instruments must be transported from the place of use to the place of reprocessing such that neither the user, third parties, the environment nor the medical devices are endangered or damaged (placement in closed, puncture-proof containers and - if necessary - use of protective caps).</p>
<p>Preparation before cleaning</p>	<p>Instruments should be reprocessed immediately after use because it is very difficult to remove dried residues from instrument parts that are difficult to access. Do not immerse in normal saline solutions (risk of pitting or stress corrosion).</p> <p>Instruments that were connected to each other during use must be disassembled into their original condition before cleaning.</p>
<p>Disassembly</p>	<p>See section 10) <i>Disassembly</i></p>
<p>Manual Pre-cleaning:</p>	<p><u>Validated procedure:</u></p> <p>Equipment: Basin Soft brush Water spray gun (or similar)</p> <p>Detergent: Neodisher® MediClean forte (Dr. Weigert)</p> <p><u>Procedure/Parameters:</u></p> <ul style="list-style-type: none"> • Rinse instruments, if possible in disassembled condition, under running cold water of potable water quality (< 40 °C) until all visible contamination has been removed. Remove stubborn contamination with a soft brush (not a wire brush). • Cavities, crevices, slits and lumens must be rinsed intensively (> 10 seconds) with cold water (potable water quality, < 40 °C) using a water spray gun (or similar). • Place the products for 10 - 30 minutes in a solution with 0.5 – 2% Neodisher® MediClean forte with water (potable water quality, < 40 °C). • Use only an approved solution of a detergent that has no protein-fixing effect. Follow the instructions of the detergent and disinfectant manufacturer. • Ensure that all areas of the instrument come into contact with the solution. • If necessary, the moving parts of the instrument may be moved back and forth in the cleaning bath. • Remove coarse contamination using a suitable brush (not a wire brush) during the exposure time. • Rinse the instruments for one minute in cold deionised water (see "<i>General Information on Reprocessing</i>") and, if applicable, move movable parts back and forth.
<p>Cleaning/ Disinfection</p>	<p>If possible, a washer/disinfector according to DIN EN ISO 15883, which uses thermal disinfection, is to be preferred.</p>



<p>Cleaning: Automated</p>	<p>Avoid overfilling instrument trays and washing trays - use only suitable instrument holders.</p> <p>When placing instruments in the sterilisation baskets and removing them afterwards, take special precautions to ensure that the tips do not become stuck in the mesh.</p> <p><u>Validated procedure:</u></p> <p>Equipment: Washer/Disinfector G 7835 CD (Miele) / PG 8535 (Miele)</p> <p>Cleaning program: Des-Var-TD (G 7835 CD)</p> <p>Detergent: Neodisher® MediClean forte (Dr. Weigert)</p> <p><u>Preparation:</u></p> <ul style="list-style-type: none"> • Instruments with joints are to be placed in the device so that the joints are opened or disassembled if possible, and the water can flow from the cavities and blind holes. • If applicable, loosen springs. • Ensure that the area inside all the cavities is also completely rinsed. • Ensure that no areas are left unwashed. • Connect the Luer connectors of the instruments, if present, to the Luer lock rinsing attachment of the washer/disinfector. <p><u>Procedure/Parameters:</u></p> <ul style="list-style-type: none"> • Pre-wash for 3 minutes with cold water (potable water quality, < 40 °C) • Emptying • Clean for 10 minutes with a solution of 0.5 – 2% Neodisher® MediClean forte in water (potable water quality) at 55 °C • Emptying • Rinse for 2 minutes with water (potable water quality, < 40 °C) • Emptying • Rinse for 1 minute with cold deionised water (< 30 °C) • Emptying • Thermodisinfection for 5 minutes with deionised water (> 90 °C) • Dry for 30 minutes (90 °C) <p>After cleaning in the machine, inspect cavities, blind holes, etc. for visible contamination. If necessary, repeat the cycle or clean manually.</p>
<p>Cleaning: Manually</p>	<p><u>Validated procedure:</u></p> <p>Equipment: Basin Soft brush Water spray gun (or similar) Bandelin Sonorex Digitec</p> <p>Detergent: Neodisher® MediClean forte (Dr. Weigert)</p> <p><u>Procedure/Parameters:</u></p> <ul style="list-style-type: none"> • Place instruments, if possible in disassembled condition, in cold water (potable water quality, < 40 °C) for 10 minutes.

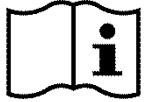

	<ul style="list-style-type: none"> • Move any movable parts, if present, back and forth over the entire range of movement. • Use a soft brush (not a wire brush) to clean the instruments until contamination is no longer visible. • Rinse the instruments for at least 20 seconds using a water spray gun (or similar). <p><u>Ultrasonic cleaning:</u></p> <ul style="list-style-type: none"> • Clean for 10 minutes at < 40 °C with 0.5 – 2% cleaning solution at 35 kHz • After ultrasonic cleaning, rinse the instruments for at least 20 seconds using a water spray gun (or similar). • Rinse the instruments for at least 10 seconds with water (potable water quality, < 40 °C). • Deionised water (< 40 °C) is to be used for the final rinse. The instruments are rinsed for at least 30 seconds with deionised water. Ensure that no residues remain on the products.
<p>Disinfection: Manually</p>	<p>Consult the instructions on the label when selecting a disinfectant (see chemical manufacturer's information).</p> <p><u>Validated procedure:</u></p> <p>Equipment: Basin Bandelin Sonorex Digitec</p> <p>Disinfectant: Korsolex® med AF (Bode Chemie GmbH)</p> <p><u>Procedure/Parameters:</u></p> <ul style="list-style-type: none"> • After cleaning, place the products in an ultrasonic bath (35 kHz, < 40 °C) with a suitable disinfectant solution (e.g. 0.5% Korsolex® med AF) for 5 minutes. Ensure that all surfaces are wetted with the disinfectant. If applicable, move the moving parts in the disinfection bath before switching on the ultrasonic cleaner. • After disinfection, rinse all products thoroughly with deionised water (< 40 °C) for at least 1 minute to remove the disinfectant and, if applicable, move the moveable parts of the instrument back and forth. • Ensure that no residues remain on the products. • Dry with sterile, oil-free compressed air.
<p>Drying</p>	<p>If drying is to be achieved as part of the cleaning/disinfection cycle, do not exceed 120 °C. Then dry with suitable compressed air in accordance with Robert Koch Institute (RKI) recommendations. Pay particular attention to the drying of difficult-to-access areas.</p>
<p>Assembly</p>	<p>See section 9) <i>Assembly</i></p>
<p>Maintenance, inspection and testing</p>	<p>For instruments with movable components that are exposed to friction (e.g. joints), an instrument oil based on paraffin/white oil (according to the valid European or United States Pharmacopoeias) which is biocompatible, steam sterilisable and steam-permeable is to be applied before sterilisation. Such places are additionally marked by a corresponding symbol of an oil can.</p>



	<p>Instruments must not be treated with care products containing silicone. These can lead to stiffness and compromise the effect of steam sterilisation. Perform a safety check of the instruments before each use. When doing so, check for sharp edges, cracks, fractures and mechanical malfunctions and missing components.</p> <p>Check instruments with movable parts for smooth operation (avoid excessive looseness). Check locking mechanisms, if applicable.</p> <p>All instruments: use a magnifying lamp to visually inspect the components for damage and wear and tear.</p> <p>In particular, inspect the critical points on moving parts and in the working area.</p> <p>Defective or damaged instruments, or those with illegible markings, must be sorted out and cleaned and disinfected before being returned to the manufacturer. Repairs may only be carried out by the manufacturer or by workshops authorised by the manufacturer. A verification form for this process is available from the manufacturer.</p> <p>Instruments that can no longer be repaired must be disposed of as scrap metal in accordance with hospital practice. In the case of surgical instruments with tips or sharp edges in particular, safe storage in a closed, puncture and break-proof disposable container must be ensured. Do not use damaged instruments.</p>
<p>Packaging</p>	<p>Individually: in accordance with the DIN EN 868 series, DIN EN ISO 11607 and DIN 58953.</p> <p>Sets: Sort instruments into dedicated trays or place them in general-purpose sterilisation trays. Pack the trays appropriately using a suitable procedure.</p>
<p>Sterilisation</p>	<p>Steam sterilisation in a fractionated vacuum process in a device complying with DIN EN 285 and DIN EN ISO 17665 (Parts 1 and 2). In order to prevent staining and corrosion, the steam must be free of contaminants. The recommended limits for contaminants for feed water and steam condensate are defined by DIN EN 285.</p> <p><u>Validated procedure:</u></p> <p>Equipment: Tuttnauer autoclave Type B 3870 EHS / Lautenschläger ZentraCert</p> <p><u>Procedure/Parameters:</u></p> <p>Cycle type: 3 pre-vacuum phases</p> <p>Sterilisation temperature: 132 – 134 °C</p> <p>Holding time: 4 – 5 minutes</p> <p>Drying time: 20 minutes</p> <p>When sterilising more than one instrument in a sterilisation cycle, do not exceed the maximum load of the steriliser (see manufacturer's instructions).</p>



Storage	<p>In accordance with § 4 MPBetreibV (Medical Devices Operator Ordinance) and the standards of the DIN EN 868 series, DIN EN ISO 11607 and DIN 58953.</p> <p>Instruments must be stored dry, at room temperature, clean, protected from damage and mechanical influences (avoid condensation, damage). Always keep instruments, if applicable, in a released state. This counteracts premature fatigue of the spring tension.</p> <p>Instruments must be transported to their place of use in a closed, puncture-proof sterile container.</p>
Disposal	<p>These products largely consist of steel or titanium. They are to be cleaned before disposal. They can be disposed of at a scrap metal recycling facility. To protect employees, care must be taken to ensure that any pointed tips or sharp edges are protected.</p>
<p>The instructions listed above were validated by the medical device manufacturer as suitable for preparing a medical device for reuse. It is the responsibility of the reprocessor to ensure that the reprocessing actually performed using equipment, materials, and personnel in the reprocessing facility achieves the desired result. This requires verification and/or validation and routine monitoring of the process. Likewise, any deviation by the reprocessor from the provided instructions should be carefully evaluated for efficacy and potential adverse consequences.</p>	
	<p>Any modification to the device or any deviation from these Instructions for Use will result in exclusion of liability. Subject to change without notice.</p>

7) Configuration and application

Clamps generally consist of two branches connected by a joint. The clamping action of the distal ends can be produced in two ways:

- Two branches with proximal handles are pressed together via a pivot point, thereby moving the branch parts towards each other at the distal end. The clamp is locked by means of a locking mechanism.
- The clamp is held in the closed state by spring force (by crossed branches or coil compression spring). Pressing the branches together spreads the distal ends apart.

Due to the variety of possible anatomical and physiological conditions, the clamps differ in their specific properties, such as overall length, length and shape of the branches, length and design of the working end and design of the handles, etc.

	<p>Bulldog clamps marked with "V" are clamps for veins and only withstand weak pressure. Bulldog clamps marked with "A" are clamps for arteries and withstand strong pressure.</p>
	<p>Use only sterilised products of sound quality.</p>
	<p>Before using the clamps, ensure that the surgical field has been prepared accordingly beforehand.</p>
	<p>Medical devices made of ferromagnetic materials must not be exposed to either a magnetic field or external electromagnetic influences.</p>



	Medical devices containing metals are electrically conductive and must not be exposed to a power source or external electrical influences.
	The choice of clamps depends on the anatomical and physiological conditions as well as the field of application. Care should be exercised to ensure that the clamps used have the correct size and geometry as well as sufficient stability.
During application	
	Incorrect use can lead to permanent deformations, such as bending of the branches or the jaw, and may even result in the breakage of these parts.
	Risk of breakage due to overloading. Risk of injury.
	Do not use clamps to grasp and/or hold objects that are too hard or too large.
	Always keep micro-instruments separate from other instruments on the operating table.

8) Required accessories

No accessories are required for using the clamps.
The clamps are stand-alone instruments. Combination with other products is therefore not intended. The exception is bulldog clamps and vascular clips. These can, under certain circumstances, also be applied to the respective tissue using an applicator forceps.

9) Assembly


No assembly of the clamps is necessary.

10) Disassembly

No disassembly of the clamps is necessary.

11) Obligation to report serious incidents

The user is obliged to report serious incidents which have occurred in connection with the medical device to the manufacturer either by e-mail to vigilance@fehling-instruments.de or via the complaint form at <https://www.fehling-instruments.de/en/complaint/> and to the competent authority of the Member State in which the user is domiciled.

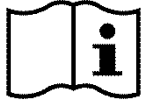

Symbols		
Where shown on the medical device, medical device label or instructions for use, the symbols have the following meanings according to DIN EN ISO 15223-1:		
 Manufacturer	 Consult instructions for use or consult electronic instructions for use	 Caution
<div style="border: 1px solid black; padding: 2px; display: inline-block;">REF</div> Catalogue number	<div style="border: 1px solid black; padding: 2px; display: inline-block;">LOT</div> Batch code	<div style="border: 1px solid black; padding: 2px; display: inline-block;">SN</div> Serial number
<div style="border: 1px solid black; padding: 2px; display: inline-block;">MD</div> Medical device	<div style="border: 1px solid black; padding: 2px; display: inline-block;">UDI</div> Unique device identifier	 CE marking
 Oil can for points to be lubricated	 CE marking	
Manufacturer's contact information		 CE marking
	FEHLING INSTRUMENTS GmbH Seligenstädter Str. 100 63791 Karlstein, Germany Tel.: +49 (0) 6188-9574-40 Fax: +49 (0) 6188-9574-45 E-mail: info@fehling-instruments.de www.fehling-instruments.de	