


FEHLING APART PUNCHES

1) Components / Description

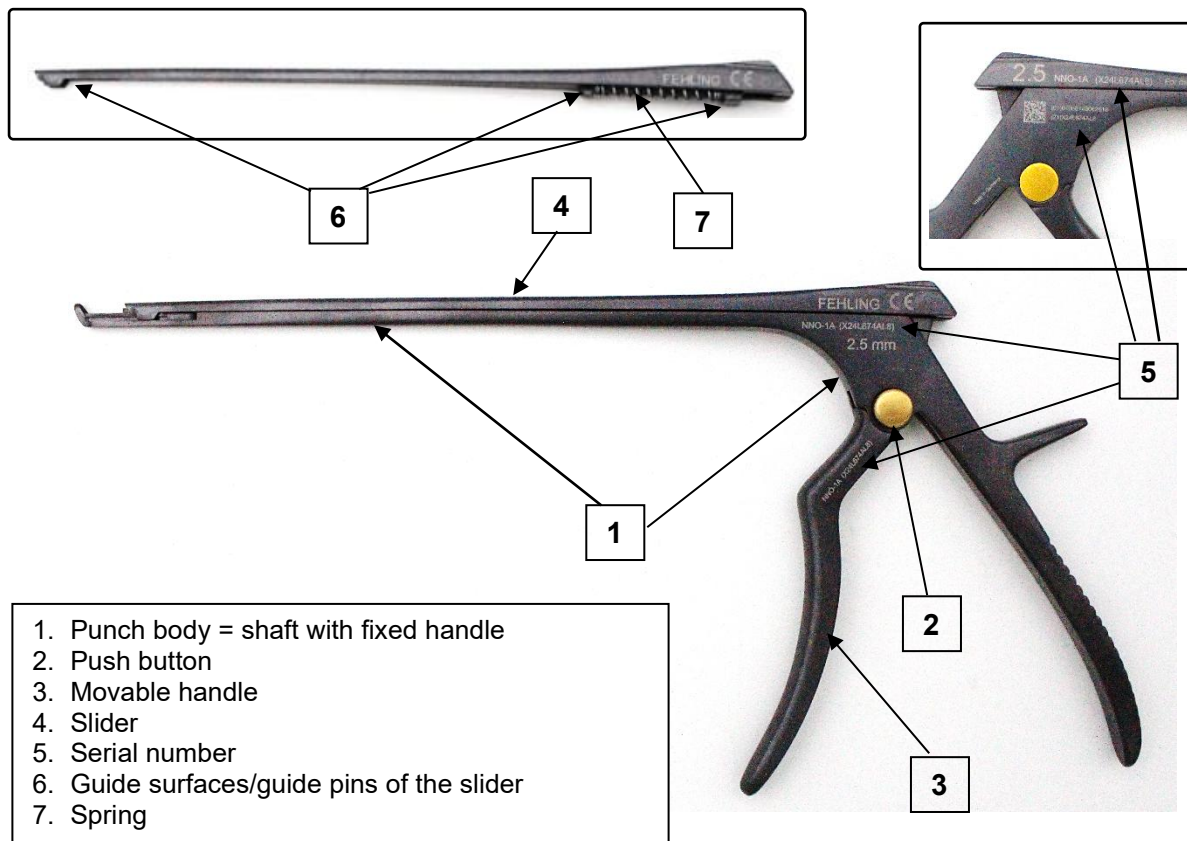

2) Disassembly

1. Hold the punch in both hands as shown.



2. Press the push button (2) with the thumb of your right hand and simultaneously place the index finger of your right hand on the slider (4) (to prevent the slider (4) from falling out unintentionally).



- When the push button (2) can no longer be depressed any further, hold the position and simultaneously pull the movable handle (3) diagonally downwards with your left hand.



The movable handle (3) cannot be pulled out completely as it is connected to the punch body (1) by the push button (2).




- The slider (4) is now released from its holder and can be easily removed.
- Always reprocess the two individual parts together in the same container.



The punch body (1), the movable handle (3) and the slider (4) have an identical serial number for verification (5), for example X19E214WM. Only instrument parts with the same serial number (5) may be assembled.

3) Assembly

- Place the slider (4) with the spring (7) onto the shaft (1). Insert the guide surfaces/guide pins (6) of the slider (4) into the corresponding receptacles in the shaft (1).

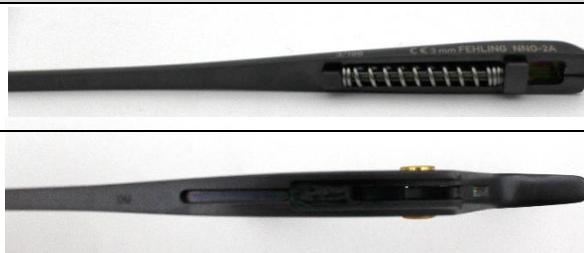

The guide surfaces/guide pins (6) of the slider (4) must fit into the corresponding receptacles in the shaft of the punch body (1).

- Slowly press the slider (4) towards the footplate until the end of the slider (4) extends approximately 5 mm beyond the end of the shaft towards the footplate.



3. While maintaining the position of the slider (4) and shaft (1) unchanged, slide the movable handle (3) into the punch body (1) until it audibly clicks into place.



4. The movable handle (3) is only correctly engaged if the push button (2) is completely extended as shown. If the position of the push button appears as shown in the "INCORRECT" illustration, assembly must be repeated. To do this, first disassemble the instrument as described in Section 2) *Disassembly*, then reassemble it as described in Section 3) *Assembly* and verify the position of the push button again.



CORRECT

INCORRECT

	The punch may only be used if the movable handle (3) is correctly locked!
	The golden locking button must never be pressed during use. This could cause the punch to completely or partially disassemble.
	Especially when using considerable force, make sure that the thumb or any other part of the hand does not press on the locking button. The pressure could trigger the unlocking mechanism.
	If the punch catches or does not run smoothly in the guide, immediately check that it has been assembled correctly. A correctly assembled punch can be recognised when the golden locking button on the labelled punch side protrudes completely.
	If this is not the case, the punch must be handed over to the surgical staff for proper assembly before further use.
5. After a functional test, the punch can be used again.	

Symbols		
Where shown on the medical device, medical device label or instructions for use, the symbols have the following meanings according to DIN EN ISO 15223-1:		
 Manufacturer	 Consult instructions for use or consult electronic instructions for use	 Caution
 Catalogue number	 Batch code	 Serial number
 Medical device	 Unique device identifier	
 Oil can for points to be lubricated	 CE marking	
Manufacturer's contact information		
	FEHLING INSTRUMENTS GmbH Seligenstädter Str. 100 63791 Karlstein, Germany Tel.: +49 (0) 6188-9574-40 Fax: +49 (0) 6188-9574-45 E-mail: info@fehling-instruments.de www.fehling-instruments.de	